THERA|CARE

Deep Tissue Massager

Model 19-016

**PLEASE READ THESE INSTRUCTIONS
CAREFULLY TO ENSURE PROPER USE AND
SAFE OPERATION OF YOUR MASSAGER.**

SAVE THESE INSTRUCTIONS

INSTRUCTION MANUAL • ENGLISH

INDEX

Introduction.....	2
Warnings & Cautions	3-4
Features of Your Deep Tissue Massager	5
How to Use Your Deep Tissue Massager	6
Care, Maintenance and Storage.....	6
Electrical Specifications.....	7
FCC Information.....	7
Warranty	8
Instrucciones en Español.....	9-16

Toll-Free Customer Care Help Line: 1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST

Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1564 11/22
©2022 Veridian Healthcare, LLC

INTRODUCTION

Thank you for purchasing this Deep Tissue Massager. Please read this instruction manual completely before using your unit to ensure safe and effective use, as well as long life of the product.

Personal Massagers are intended for personal comfort, including temporary relief of minor aches and pains. Not intended for medical use.

WARNINGS & CAUTIONS

**PLEASE READ THIS INSTRUCTION MANUAL
COMPLETELY BEFORE OPERATING THIS DEVICE.**

**DANGER- TO REDUCE THE RISK OF BURNS, ELECTRIC SHOCK
AND FIRE, THIS PRODUCT MUST BE USED IN ACCORDANCE
WITH THE FOLLOWING INSTRUCTIONS.**

IMPORTANT SAFETY INSTRUCTIONS

When using your electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS AND SAFEGUARDS BEFORE USING THIS MASSAGER.
KEEP AWAY FROM WATER; DO NOT USE WHEN IN WATER.

DANGER

To reduce the risk of death or injury by electric shock:

1. Do not reach for an appliance that has fallen into water. Unplug immediately.
2. Do not use while bathing or in the shower.
3. Do not place or store appliance where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid.
4. Always unplug this appliance from the electrical outlet immediately after using and before cleaning.
5. Do not use massager without attachments.

WARNING

To reduce the risk of burns, electrical shock or injury to persons:

1. This appliance should never be left unattended when plugged in. Unplug from outlet when not in use, and before putting on or taking off massage heads.
2. Do not operate this appliance under a blanket or pillow. Excessive heating can occur and cause fire, electric shock or injury to persons.
3. Close supervision is necessary when this appliance is used by, on, or near children or disabled persons.
4. This massager contains an electro magnetic motor. To avoid risk of injury, it must not be used within six inches of an implantable cardiac device.
5. Use this appliance only for its intended use, as described in this manual. Do not use attachments not recommended by the manufacturer.

WARNINGS & CAUTIONS


6. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.
7. Do not carry this appliance by electric cord or use cord as a handle.
8. Loop cord loosely when storing, tight wrapping may damage cord and internal parts.
9. Keep the cord away from heated surfaces.
10. Never drop or insert any object into any opening on this appliance.
11. Do not use outdoors.
12. Do not use where aerosol products are being used or where oxygen is being administered.
13. In case of pregnancy, diabetes or illness, consult your physician before using this massager.
14. Do not use massager if you have any of the following: sensitive skin, swollen or inflamed areas, in the presence of poor circulation, on insensitive skin, where skin eruptions are present, in the presence of unexplained calf or abdominal pain, varicose veins, front of the neck, genital areas.
15. Do not use damaged attachments. Discontinue use of attachments if cracked or broken. May cause injury to skin.
16. To disconnect, turn all controls to the OFF position, then remove plug from outlet.
17. Do not use massager in close proximity to loose clothing or jewelry.
18. Keep long hair away from massager while using.
19. Use heat feature carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

FEATURES OF YOUR DEEP TISSUE MASSAGER

MESSAGE HEAD ATTACHMENTS

This massager includes 7 massage head attachments for a variety of uses and massage experiences. Select the best head for your personal comfort preference.

			
<p>ROUND MESSAGE HEAD General head for whole body use.</p>	<p>HEAT MESSAGE HEAD Wide dome head for use with heat function.</p>	<p>PINPOINT MESSAGE HEAD Soft, silicone nodes for deep muscle relaxation.</p>	<p>FOOT MESSAGE HEAD Short nub silicone nodes for use on the feet.</p>
			
<p>ESSENTIAL OIL MESSAGE HEAD Application head for applying essential oil or cream.</p>	<p>6-NODE MESSAGE HEAD Deep, penetrating massage for back and waist.</p>	<p>SHIATSU POINT MESSAGE HEAD Nodes break up muscles, knots and adhesion for relief at specific trigger points. Suited for lower back, soles of feet, shoulders, neck.</p>	

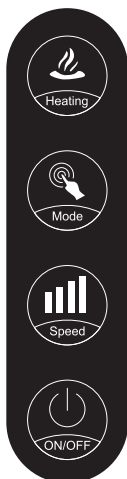DIGITAL DISPLAY AND BUTTONS

DIGITAL DISPLAY — Indicates Power ON / OFF, LEVEL for all features and remaining TIMER.

TIMER — Automatic timer default to 15 minutes; massager will turn off after 15 minutes. After setting preferences, display will show remaining timer countdown.

HEATING — For use only with Heating Massage Head; head will illuminate red to show it is on.

MODE — 6 massage modes: A1, A2, A3, A4, A5, A6

SPEED — 6 massage speeds: S1, S2, S3, S4, S5, S6

ON/OFF — Power button

HOW TO USE YOUR DEEP TISSUE MASSAGER

Read the **WARNINGS & CAUTIONS** section of this manual before use and always use common sense care when operating any electrical device.

EASY STEP INSTRUCTIONS

1. Plug the product into a standard electrical outlet.
2. Select and apply the desired massage head. Take care the head is applied firmly.
3. Use the controller to set the desired massage strength and massage mode. Press and release Mode and Speed to scroll through settings; start on the lower setting until you have adjusted to the strength of the massager.
4. If using the wide heat massage head, press the 'Heating' button to activate the heat feature. The heat massage head will illuminate red while in use.
5. The massager will turn off after 15 minutes; a countdown will show on the digital display.
6. Always unplug the device when not in use. Store away from children.

SAFE FOR USE ON: ARMS, SHOULDERS, UPPER AND LOWER BACK, LEGS, FEET
DO NOT USE ON: FACE, FRONT OF NECK OR GENITALIA

CARE, MAINTENANCE AND STORAGE

DISCONNECT THE MASSAGER FROM ELECTRICAL OUTLET AFTER USE AND BEFORE CLEANING.

- Power OFF and unplug massager when not in use.
- Do not submerge in water.
- Massager body – to clean, wipe with a soft cloth; do not use water, do not submerge, do not use soap or other chemical cleaners.
- Massager heads – all heads except 'heat massage head' may be cleaned with warm soap and water, rinse thoroughly and allow to dry completely before use; heat massage head may be wiped with a soft cloth, do not get wet or submerge.
- Store in a cool, dry place. Avoid exposure to prolonged sunlight. Do not store in the bathroom or other humid environments.

ELECTRICAL SPECIFICATIONS

VOLTAGE	AC 110 - 120V
FREQUENCY	50 Hz
POWER	25W

FCC INFORMATION

IMPORTANT INFORMATION REQUIRED BY THE FCC

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference; and (2) this device must accept any interference received, including interference that may cause undesirable operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

CAUTION: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

1-YEAR LIMITED WARRANTY

The warrantor guarantees that its Deep Tissue Massager will be free from manufacturing defects under normal use for a period of one-year. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty is voided if the unit is misused or abused in any manner.

If the Deep Tissue Massager fails to conform to this limited warranty, return the Heating Pad postage prepaid to: **Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**. When returning a product, please also include: (i) a copy of your receipt, bill of sale or other comparable proof of purchase; (ii) a written description of the problem; and (iii) your name, address and telephone number. Carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. At our option, the warrantor will repair or replace the unit found to be defective in materials or workmanship under normal consumer usage. The purchaser will be notified of any additional repairs required prior to completing the repair, and will be responsible for parts charges, if any, and repair charges not covered by this limited warranty.

EXCEPT AS PROVIDED FOR IN THIS LIMITED WARRANTY, ALL EXPRESS AND IMPLIED WARRANTIES AND CONDITIONS ARE DISCLAIMED, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL THE WARRANTOR BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR DAMAGES TO, OR LOSS OF, OTHER PROPERTY OR EQUIPMENT OR PERSONAL INJURIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

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